









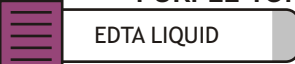
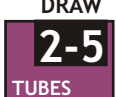
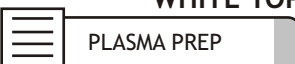



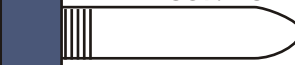
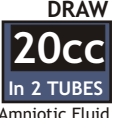


TUBE SELECTION

TUBE TYPE	TEST TYPE	HANDLING INSTRUCTIONS
 SWAB KIT	OBTAIN (1)  Use For: <ul style="list-style-type: none"> Fetal Fibronectin (fFn) Preterm 	Handling: <ul style="list-style-type: none"> Vaginal swab patient. One swab kit per person. Refrigerate until shipment
 DNA SWAB KIT DNA KIT	OBTAIN (1)  Use For: <ul style="list-style-type: none"> Paternity Testing 	Handling: <ul style="list-style-type: none"> Swab firmly inside of each patient's mouth area. One swab kit per person. Refrigerate until shipment
 GOLD or RED/GREY TOP SERUM SEPARATOR	DRAW  Use For: <ul style="list-style-type: none"> Combined Test Integrated 190 Test Modified Sequential 30•190 Test Contingent 30•1500•190 Test QUAD Test Triple Test Serum NTD only 	Handling: <ul style="list-style-type: none"> Allow to clot about 20 minutes and spin 2500 rpm for 10 minutes Refrigerate until shipment
 DARK GREEN TOP SODIUM HEPARIN	DRAW  Use For: <ul style="list-style-type: none"> Peripheral Blood Chromosome Analysis AneuVysion[®] FISH 	Handling: <ul style="list-style-type: none"> Draw full tube Invert tube 8 times to prevent clotting Refrigerate until shipment
 BLUE TOP SODIUM CITRATE	DRAW  Use For: <ul style="list-style-type: none"> Tay Sachs (Enzyme) 	Handling: <ul style="list-style-type: none"> Draw 4 full tubes Invert tube 8 times to prevent clotting Refrigerate until shipment
 PURPLE TOP EDTA LIQUID	DRAW  Use For: <ul style="list-style-type: none"> Cystic Fibrosis Tay-Sachs (DNA) Bloom Syndrome Canavan Disease Gaucher Disease Familial Dysautonomia Fragile X Syndrome Sickle Cell Anemia Mucopolipidosis (Type IV) Fanconi Anemia (Group C) Glycogen Storage (Type 1A) Maple Syrup Urine Disease Nieman-Pick Disease (Type A & B) Y Chromosome Microdeletion Ashkenazi Jewish Carrier 9 Screen 	Handling: <ul style="list-style-type: none"> Draw 2 to 5 full tubes, depending on the number of tests ordered. Invert tube 8 times to prevent clotting Refrigerate until shipment
 WHITE TOP PLASMA PREP	DRAW  Use For: <ul style="list-style-type: none"> RhD Test SRY, X & Y Test 	Handling: <ul style="list-style-type: none"> Draw 3 White and 1 Purple tubes. Invert tube 8 times to prevent clotting Refrigerate until shipment MUST DRAW & SHIP MONDAY-THURSDAY ONLY! DO NOT FREEZE
 PURPLE TOP EDTA LIQUID	DRAW  	
 FALCON TUBE	DRAW  Use For: <ul style="list-style-type: none"> Prenatal Chromosome Analysis Amniotic Fluid AFP QF-PCR (Chromosomes 13, 18, 21, X, Y) Chorionic Villus Sampling (CVS) 	Handling: <ul style="list-style-type: none"> Maintain room temperature DO NOT FREEZE